

Cycler Instructions for Inpatient Pediatric Unit Nurses

Programming Cycler – Standard Mode

Must be done before starting set-up

YouTube Video Link

Baxter International

HomeChoice / HomeChoice Pro APD System Clinician Training Video

https://youtube.com/playlist?list=PLxOc1MKY05MOD0Ap8jxa1Bz_crk--b5IH&si=gfORfOKQHWcHECm8

Video # 12 – Hospital or Acute Settings/Standard Mode

Programming Cycler @ 5:15-11:01 (**DO NOT TURN CYCLER OFF**)

Programming Cycler – Low Fill Mode

Must be done before starting set-up

YouTube Video Link

Baxter International

HomeChoice / HomeChoice Pro APD System Clinician Training Video

https://youtube.com/playlist?list=PLxOc1MKY05MOD0Ap8jxa1Bz_crk--b5IH&si=gfORfOKQHWcHECm8

Video # 11 – Low Fill Mode until 16:47 (**DO NOT TURN CYCLER OFF**)

Set-Up

YouTube Video Link

Baxter International

HomeChoice / HomeChoice Pro APD System Clinician Training Video

https://youtube.com/playlist?list=PLxOc1MKY05MOD0Ap8jxa1Bz_crk--b5IH&si=gfORfOKQHWcHECm8

Video # 12 – Hospital or Acute Settings/Standard Mode

Connecting

YouTube Video Link

Baxter International

HomeChoice / HomeChoice Pro APD System Patient Training Video

https://www.youtube.com/playlist?list=PLxOc1MKY05MM462qbeQZwIspYddk_3z2Z

Video # 12 (Prepare for Your Therapy) from 18:41 – 19:20

Troubleshooting

YouTube Video Link

Baxter International

HomeChoice / HomeChoice Pro APD System Clinician Training Video

https://youtube.com/playlist?list=PLxOc1MKY05MOD0Ap8jxa1Bz_crk--b5IH&si=gfORfOKQHWcHECm8

Video # 24 – Troubleshooting and Alarms/Check Therapy Valve

Video # 27 – Low UF

Video # 28 – Low Drain Volume

Video # 30 – Verify I-Drain

Video # 31 – Caution: Negative UF

Video # 32 – Check lines

Video # 33 – System Error 2240 or 2267

Video # 36 – Power Failure

End Therapy

YouTube Video Link

Baxter International

HomeChoice / HomeChoice Pro APD System Patient Training Video

https://www.youtube.com/playlist?list=PLxOc1MKY05MM462qbeQZwlSpYddk_3z2Z

Video # 14 (End Therapy)

Completing treatment & gathering data @ 0:20 to 1:29

Disconnection @ 2:34-3:46

Finish & clean up @ 3:49-4:44